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
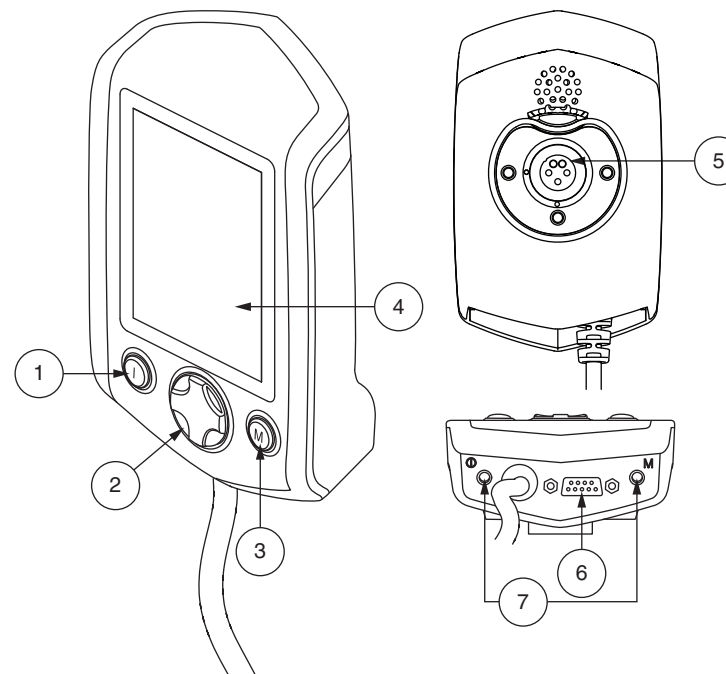
# Q-LOGIC 2 ENHANCED DISPLAY 2.0 OWNER'S MANUAL UPDATE

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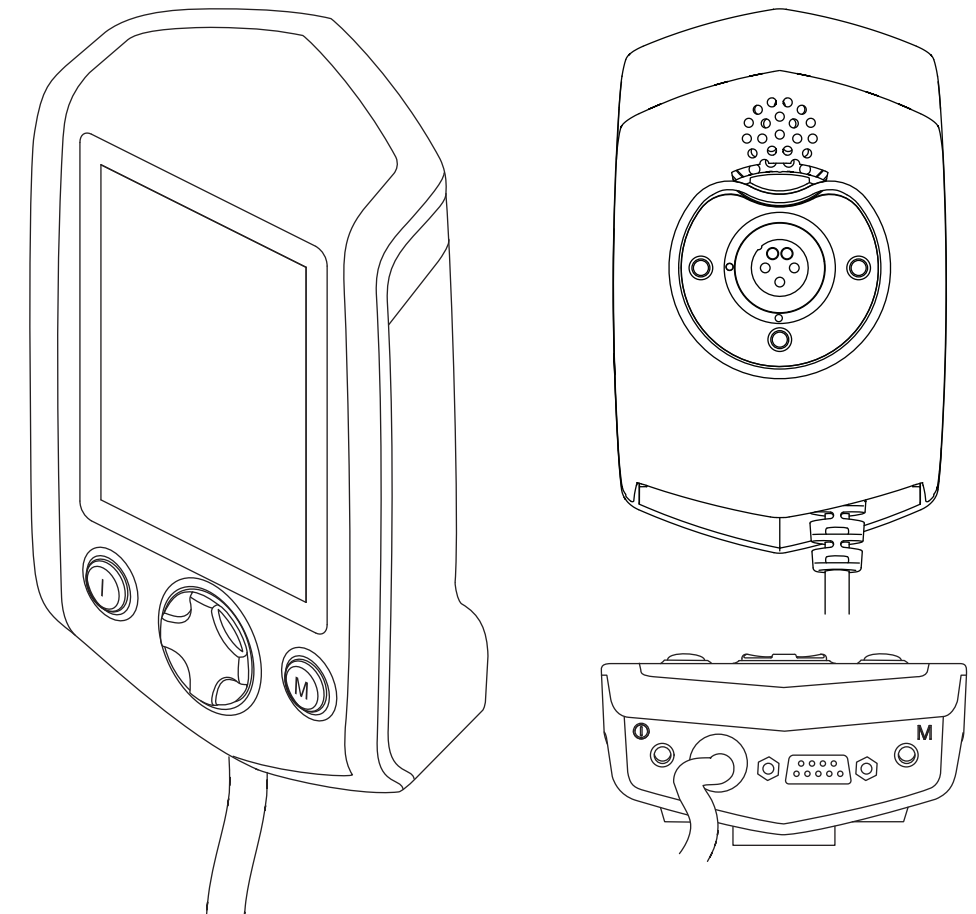## Q-LOGIC 2 ENHANCED DISPLAY 2.0 OWNER'S MANUAL UPDATE

### Enhanced Display

The Enhanced Display module is primarily used with specialty control devices like a head array. It also can be used to provide a graphical display for the Stand-alone Joystick. When used with the hand control, it provides a larger display than the hand control's LCD. The Enhanced Display is mainly used in more advanced applications. Contact your Quantum Rehab Provider for more information.

The Enhanced Display has the following controls:

- **On/Off Button**  
Enables/disables power
- **On/Off and Mode Jacks**  
Allow for remote On/Off and Mode switch installations
- **Mode Button**  
Allows changes to client modes without using the client's switch or input device
- **9-pin D-sub Connector**  
Allows for connection of a specialty control device.
- **LCD Screen**  
A color LCD display that may be equipped with built-in infrared (IR) for remote control and with Bluetooth mouse control
- **4-Direction Navigation Key**  
The up/down arrows of the four-direction navigation key activate seating functions while the left/right arrows select previous (left) or next (right) seating functions (multiple seating functions required). The up/down arrows also navigate up and down through the menu on the menu screen, and the left/right arrows function as Esc command (left) and Select (right).

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