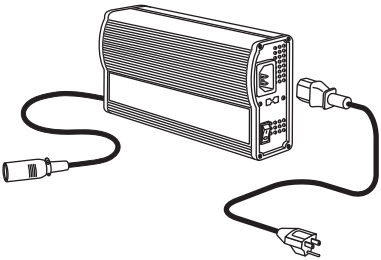
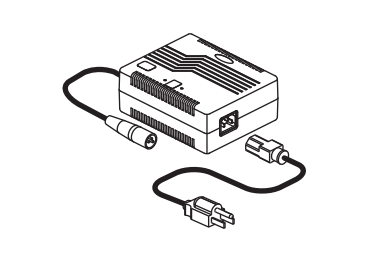
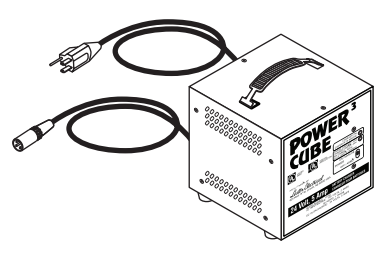
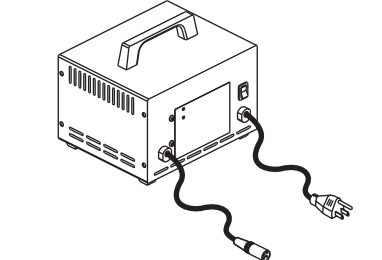
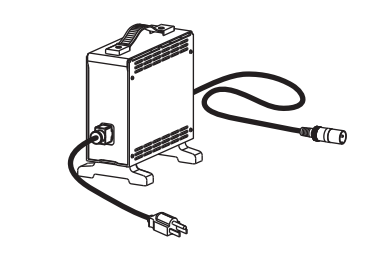
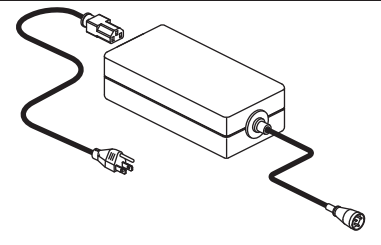
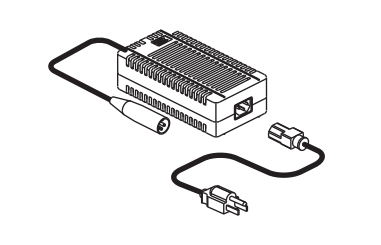
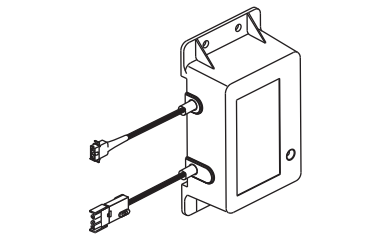
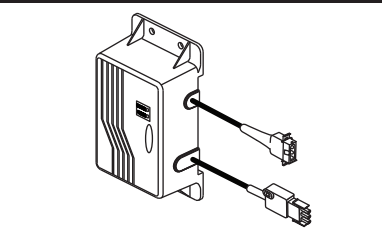
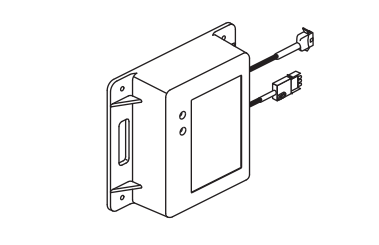


QUICK REFERENCE GUIDE: Charger Indicator Codes

5-AMP MERITS CHARGER (ELECHG1000)	5-AMP BATTERY MAXIMIZER (ELEASMB6293)	5-AMP POWER CUBE (ELE110V1044)	8-AMP LESTER (ELE110V1041)	8-AMP CTE (ELEASMB5669)
				
<p>LED INDICATIONS:</p> <p>Power Light Red: Power is on Off: Disconnected</p> <p>Status Light Yellow: Charging Green: Full Charge Off: Disconnected Red: Fault</p>	<p>LED INDICATIONS:</p> <p>Power Light Green: A/C Connected Red: Error (Short Circuit, Reversed Leads)</p> <p>Status Light Red: Fast Charging Green: Full Charge</p>	<p>LED INDICATIONS:</p> <p>Power Light Red: Power is on Off: Disconnected</p> <p>Status Light Flashing Green: 80% Charged Solid Green: Full Charge</p>	<p>LED INDICATIONS:</p> <p>Red: Fault Slow Green Blink: Under 80% Charged Fast Green Blink: Over 80% Charged Solid Green: Full Charge</p>	<p>LED INDICATIONS:</p> <p>Green Flash: Power On Red Flash: Error Orange: Charging Orange Flash: Less than 80% Charged Green/Orange Flash: 80% Charged Green: Full Charge</p>
2-AMP OFF-BOARD CHARGER (OLD: ELECHG1001) (NEW: ELECHG1007) or (ELECHG1015)	3-AMP OFF-BOARD CHARGER (ELEASMB5713)	4-AMP GUEST ONBOARD (OLD: ELE110V1029)	4-AMP BATTERY MAXIMIZER ONBOARD (POWER CHAIR: ELECHG1018) (SCOOTER: ELECHG1019)	5-AMP GUEST ONBOARD (ELE110V1037)
				
<p>LED INDICATIONS:</p> <p>Power Light Green: A/C Connected Red: Error, Short, Voltage Too Low or Blown Fuse</p> <p>Status Light Red: Fast Charging Green: Full Charge Off: Disconnected, Battery Voltage Too Low or Blown Fuse</p>	<p>LED INDICATIONS:</p> <p>Power Light Green: A/C Connected Red: Error (Short Circuit, Reversed Leads)</p> <p>Status Light Red: Fast Charging Green: Full Charge Off: Disconnected Batteries or Voltage Too Low</p>	<p>LED INDICATIONS:</p> <p>Green: Full Charge Extended Green/No Green LED: Fault (after more than 24 hours)</p>	<p>LED INDICATIONS:</p> <p>Power Light Green: A/C Connected Red: Error (Short Circuit, Reversed Leads)</p> <p>Status Light Red: Fast Charging Green: Full Charge</p>	<p>LED INDICATIONS:</p> <p>Red: Charging Red and Green: 80% Charged Green: Full Charge Extended Green/No Green LED: Fault (after more than 24 Hours)</p>