

DEDICATED TO REVOLUTIONIZING
THE POWER CHAIR EXPERIENCE



BRYAN ANDERSON

Iraq War Veteran
and Quantum Rehab®
Spokesperson

QUANTUM® OUTBACK

FDA Class II Medical Device



quantumrehab.com



5	INTRO
6	iLEVEL®
8	OUTBACK
10	EDGE® 3
12	STRETTO
14	R-TRAK
16	4FRONT® 2
20	BARIATRIC LINE
22	TRU-BALANCE® 4
24	SEATING OPTIONS
26	STEALTH PRODUCTS® CUSHIONS
30	ELECTRONICS AND ACCESSORIES
34	COLOR OPTIONS
36	APPS
38	SERVICE AND SUPPORT
40	SPECIFICATIONS





Quantum® Power Chairs are
FDA Class II Medical Devices
designed to aid individuals with
mobility impairments.



FDA Class II
Medical Device



FDA Class II
Medical Device





WELCOME TO QUANTUM®

Quantum Rehab® was born out of the desire to delight customers with the most advanced, consumer-inspired complex rehab power wheelchairs and related technologies possible.

Clinically-focused and consumer-inspired, Quantum® products are engineered and manufactured to meet the widest range of complex rehab needs with functional, durable, stylized, high-performance designs to increase daily functionality and quality of life. At Quantum, consumer needs and wishes are the driving force. We're dedicated to not just meeting medical and clinical needs, but also quality-of-life needs. From the most advanced power seating for pressure management to USB ports, Bluetooth® and fender lights, no consumer need is overlooked.

Headquartered in Duryea, Pennsylvania, Quantum also has operations in Australia, Canada, China, Italy, France, Germany, the Netherlands, New Zealand, and the United Kingdom.

quantumrehab.com



Quantum® Power Chairs are
FDA Class II Medical Devices
designed to aid individuals with
mobility impairments.

FDA Class II
Medical Device





Safer transfers



Improved reach



FDA Class II
Medical Device

"COOKING IS SO MUCH EASIER NOW THAT I CAN REACH THINGS. I CAN'T IMAGINE MY LIFE WITHOUT iLEVEL."

STEPHANIE WOODWARD
ROCHESTER, NY



FDA Class II
Medical Device

Social
interaction
at iLevel®

FDA Class II
Medical Device

FDA Class II
Medical Device

iLEVEL®

POWER SEAT ELEVATION

Standing Height + Speed + Stability = Authentic iLevel (like no other)

iLevel® power seat elevation technology features a revolutionary new power chair design for a better experience. iLevel improves daily functionality and quality of life by allowing operation of the power chair with the seat elevated while at walking speed.

FEATURES

- 12" of power adjustable seat height while at walking speed; maximum of 10" height on the Q6 Edge® HD
- iLevel's patented Extra Stability Technology® enhances safety for transfers, reach, and numerous other activities of daily living
- Available on the Stretto, Edge® 3 and Q6 Edge® 2.0
- Front and rear LED fender lights come standard with iLevel®



FDA Class II
Medical Device

MAKE YOUR OWN PATH.



Available with TRU-Balance® 3
Power Positioning (shown) or
Q-Captain Seating

Optional Quantum®
backup camera

OPTIONAL
10" safe seat elevation
at up to 3.2 mph

Rear LED lights



Independent SRS
(Smooth Ride Suspension)

Q-Logic 3e Controller
with Bluetooth®

Stealth Products®
cushion featuring
Coolcore®
technology

Front LED lights

15" pneumatic
tires for superb
control over
rough terrain

The Quantum® Outback is an FDA Class II Medical Device designed to aid individuals with mobility impairments.

FRONT LED
LIGHTING

REAR LED
LIGHTING

Q-LOGIC 3e CONTROLLER WITH BLUETOOTH®



"THE OUTBACK LETS ME LIVE LIFE THE WAY I WANT. WHETHER I'M WORKING ON THE FARM OR GOING FOR A HIKE WITH MY FRIENDS, THE OUTBACK ALLOWS ME TO GO WHEREVER I WANT TO GO."

BRYAN ANDERSON
NASHVILLE, TN



QUANTUM® OUTBACK

4X4 ALL-TERRAIN

Quantum Rehab® welcomes adventurers and nature enthusiasts to unleash their wild side with the release of our cutting-edge 4x4 all-terrain power chair, the Quantum® Outback! Designed for those who dare to defy conventional limits, this rugged mobility solution promises to elevate any outdoor adventure, whether navigating wooded trails or exploring uncharted terrain.

SPECS


MAXIMUM SPEED: Up to 5 mph (6 mph with optional upgrade)

WEIGHT CAPACITY: 300 lbs. (350 lbs. with Q-Captain Seating)

TURNING RADIUS: 49.2"

POWER SEAT ELEVATION: Optional 10" safe seat elevation

COLORS

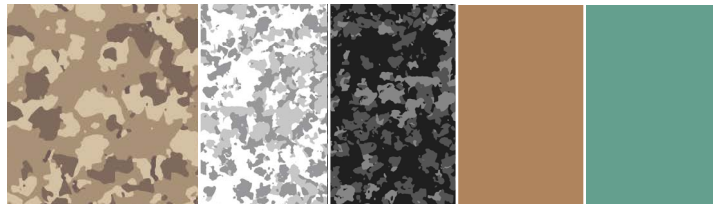

DESERT CAMO

WHITE CAMO


BLACK CAMO

TAN (MATTE)

CADET (MATTE)

AVAILABLE WITH 10"
SAFE SEAT ELEVATION

INDEPENDENT SMOOTH RIDE SUSPENSION

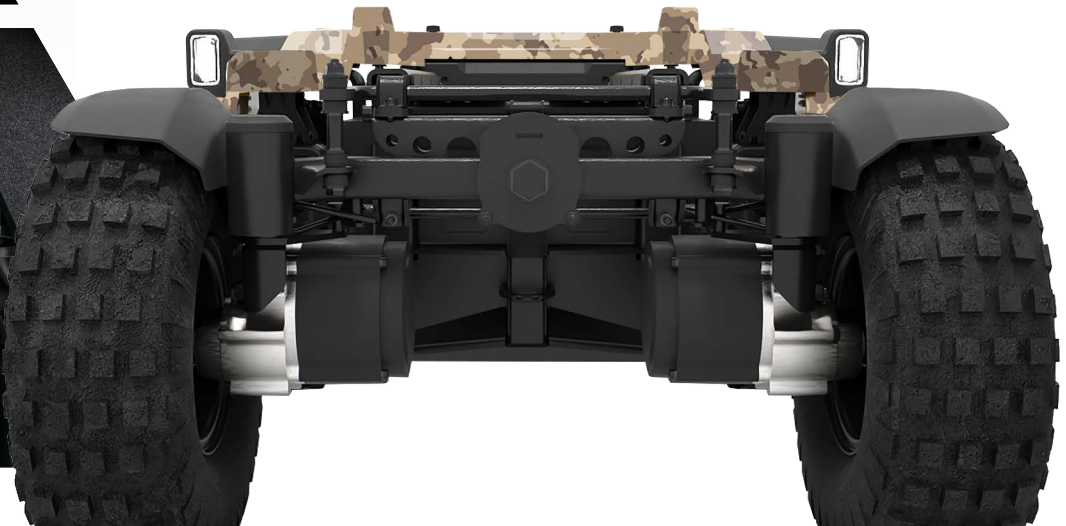

PROUDLY MADE IN
AMERICA
OF FOREIGN AND DOMESTIC PARTS



OPTIONAL

TRU-Balance® 4 Power Positioning Systems provide a choice of 10, 20, or 30 degrees of power anterior tilt.

OPTIONAL

12" iLevel® power seat elevation at 4.5 mph (standard on TRU-Balance® 4)

Featuring Extra Stability Technology® for safer transfers, reach and other activities of daily living



Q-Logic 3 Advanced Drive Controls with Bluetooth®

Stealth Products® cushion featuring exclusive Coolcore® technology

Upgraded SRS (Smooth Ride Suspension)

The Edge® 3 is an FDA Class II Medical Device designed to aid individuals with mobility impairments.

USB charger (standard)

Front and rear LED fender lights for seeing and being seen (standard)

"THE EDGE 3 WITH iLEVEL HAS RESTORED MY INDEPENDENCE AT BOTH WORK AND HOME. iLEVEL HAS ALLOWED ME TO GET MORE INVOLVED AND START A DIALOGUE WITH PEOPLE, SOMETHING I WOULDN'T HAVE DONE BEFORE."

MERLISHA HENDERSON
PHOENIX, AZ

EDGE® 3

MID-WHEEL DRIVE

The Edge® 3 offers upgraded SRS (Smooth Ride Suspension), which provides a smooth, comfortable ride and advanced stability. Optional iLevel® is available at 4.5 mph with up to 12" of power adjustable seat height.

SPECS

MAXIMUM SPEED: 6.25 mph (7 mph with optional 4-pole motors and Q-Logic 3 / 4.5 mph when elevated with iLevel®)

WEIGHT CAPACITY: 300 lbs. (See Q6 Edge® HD for increased capacity)

TURNING RADIUS: 20.5"

POWER SEAT ELEVATION: Optional 12" iLevel

COLORS

* See website for a complete list of color options.



GREEN MACHINE

RED CORVETTE

DEEP PURPLE

LEMON CRUSH

RASPBERRY BERET

EVOLUTION ORANGE

BLUE JEAN

AVAILABLE WITH iLEVEL
• POWER SEAT ELEVATION •

Drive wheel
color accents
(standard)



PROUDLY MADE IN
AMERICA
OF FOREIGN AND DOMESTIC PARTS

NARROWEST POWER
BASE IN AMERICA AT
20.47"
WIDTH WITH 12.5"
DRIVE WHEELS



OPTIONAL

TRU-Balance® 4 Power Positioning Systems provide a choice of 10, 20, or 30 degrees of power anterior tilt.

OPTIONAL

12" iLevel® power seat elevation at 3.5 mph (standard on TRU-Balance® 4)

Toolless removal of rear shroud with battery access door

Independent SRS (Smooth Ride Suspension)

Q-Logic 3 Advanced Drive Controls with Bluetooth® and USB charger

Stealth Products® cushion featuring exclusive Coolcore® technology


Front and rear LED fender lights



The Stretto is an FDA Class II Medical Device designed to aid individuals with mobility impairments.

Independent SRS (Smooth Ride Suspension)

The Stretto is the ideal pediatric power chair solution

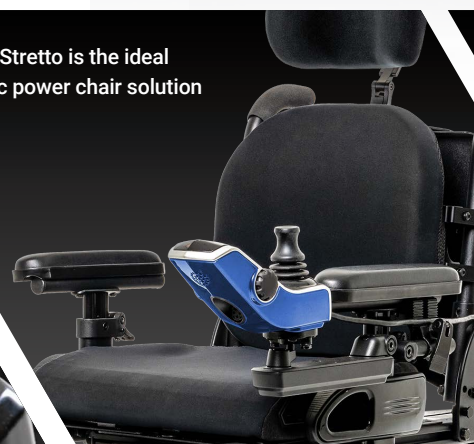

"TIGHT ROOMS, NARROW DOORWAYS, OUT-OF-REACH CABINETS AND VAN RAMP... NOW I CAN GO WHERE I WANT TO GO, ALL BECAUSE OF THE STRETTO!"

BRYSON FOSTER
CONCORD, NC

STRETTO

MID-WHEEL DRIVE

The Edge 3 Stretto® is the narrowest, most maneuverable power base in America! In addition to its narrow width, it is equipped with independent SRS (Smooth Ride Suspension), making it ideal for children, teenagers and small adults. The Stretto is available with 12.5" or 14" drive wheels and optional 12" of iLevel® power adjustable seat height.

SPECS

MAXIMUM SPEED: 6.25 mph (3.5 mph when elevated with iLevel®)

WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 21"

POWER SEAT ELEVATION: Optional 12" iLevel

COLORS

* See website for a complete list of color options.



BLUE JEAN

RED CORVETTE

DEEP PURPLE

LEMON CRUSH

RASPBERRY BERET

EVOLUTION ORANGE

GREEN MACHINE

AVAILABLE WITH iLEVEL®
POWER SEAT ELEVATION

Toolless access
to electronics



PROUDLY MADE IN
AMERICA
OF FOREIGN AND DOMESTIC PARTS

THE EVOLUTION OF REAR WHEEL DRIVE



The R-TRAK is an FDA Class II Medical Device designed to aid individuals with mobility impairments.

Independent SRS (Smooth Ride Suspension)

LED lighting in front caster arms and rear fenders



R-TRAK

As a global innovator, we are creating a product that offers confidence and stability elevated at the speed of life. Our state-of-the-art facility has a robust team of experienced technicians who custom build each R-TRAK specifically to the individual consumer's needs.



Scan to learn about the design and development of the R-TRAK.



Optional Quantum backup camera can be mounted anywhere



R-TRAK

REAR-WHEEL DRIVE

The R-TRAK revolutionizes the real-wheel drive power chair experience with independent SRS suspension on all four wheels, providing a smoother, more maneuverable ride. Standard features include fender lights, USB charger and drive wheel color accents. The R-TRAK is also available with our 10" safe seat elevation, allowing the client to drive up to 3.5 mph while elevated.

SPECS

MAXIMUM SPEED: Up to 7 mph (3.5 mph when elevated)

WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 25.7"

POWER SEAT ELEVATION: Optional 10" safe seat elevation

COLORS

* See website for a complete list of color options.



LEMON CRUSH

RED CORVETTE

DEEP PURPLE

BLUE JEAN

RASPBERRY BERET

EVOLUTION ORANGE

GREEN MACHINE





450-LB. WEIGHT CAPACITY

available on the
4FRONT® 2 HD

Optional backup camera with
3.5" display can be mounted
in multiple locations

OPTIONAL

TRU-Balance® 4 Power
Positioning Systems provide
a choice of 10°, 20°, or 30°
of power anterior tilt.

OPTIONAL

10" power adjustable safe
seat elevation at 3.5 mph
(standard on TB4)

Redesigned caster arms
for improved durability

STC (Smart
Traction Control)



Q-Logic 3e controller
featuring a magnetic
charging port (Q-Logic 3
also available)

Stealth Products®
cushion featuring
exclusive Coolcore®
technology

Front and rear LED
fender lights for
seeing and being
seen (standard)

Automotive-grade SRS
(Smooth Ride Suspension)

The 4Front® 2 is an FDA Class II Medical Device designed to aid individuals with mobility impairments.

450-lb. weight capacity is available
on the 4Front® 2 HD

Automotive-grade SRS (Smooth Ride Suspension)

"RIDING IN THE 4FRONT 2 IS UNLIKE ANY CHAIR I HAVE EVER USED. THE 4FRONT 2 PUSHES THE LIMIT SET BY ITS PREDECESSOR BEYOND MY EXPECTATIONS."

CHRISTIAN BUDNEY
SCRANTON, PA

4FRONT® 2

FRONT-WHEEL DRIVE

Featuring STC (Smart Traction Control)

The 4Front® 2 is the next generation of the 4Front® Power Chair! New Smart Traction Control (STC), along with Smooth Ride Suspension (SRS), provide stability for smooth obstacle transitions and unmatched driving performance. Additional new features include 4-pole motors and redesigned caster arms for improved durability and aesthetics and reduced weight.

SPECS

MAXIMUM SPEED: 6.25 mph (3.5 mph when elevated)

WEIGHT CAPACITY: 300 lbs. (450 lbs. on 4Front® 2 HD)

TURNING RADIUS: 25.9"

POWER SEAT ELEVATION: Optional 10" safe seat elevation

COLORS

* See website for a complete list of color options.



EVOLUTION ORANGE

RED CORVETTE

DEEP PURPLE

LEMON CRUSH

RASPBERRY BERET

GREEN MACHINE

BLUE JEAN

AVAILABLE WITH 10"
SAFE SEAT ELEVATION

LED front and rear fender lights
(front: white / rear: red)



FDA Class II
Medical Device



Q6 EDGE® 2.0

MID-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 6.25 mph (3.5 mph when elevated)


WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 20.5"

POWER SEAT ELEVATION: Optional 12" iLevel®

COLORS

* See website for a complete list.

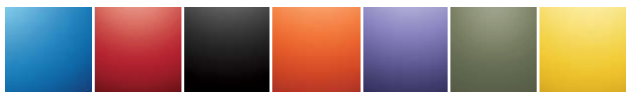

FDA Class II
Medical Device



Q6 EDGE® HD

BARIATRIC MID-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 4.7 mph (3.5 mph when elevated)

WEIGHT CAPACITY: 450 lbs. (400 lbs. with iLevel®)

TURNING RADIUS: 19.75"

POWER SEAT ELEVATION: Optional HD 10" iLevel®

COLORS

* See website for a complete list.



J4E
shown with
optional 10"
Safe Seat
Elevation

FDA Class II
Medical Device

FDA Class II
Medical Device



AVAILABLE WITH
10" SAFE SEAT
ELEVATION AND
Q4 POWER TILT

J4E

MID-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 5 mph

WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 20.5"

POWER SEAT ELEVATION: Optional 10" Safe Seat Elevation

COLORS



FDA Class II
Medical Device



AVAILABLE WITH
10" SAFE SEAT
ELEVATION AND
Q4 POWER TILT

J4

MID-WHEEL DRIVE

SPECS


MAXIMUM SPEED: 5 mph

WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 21"

POWER SEAT ELEVATION: Optional 10" Safe Seat Elevation

COLORS

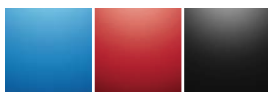

FDA Class II
Medical Device



FEATURING
Q4 HEAVY DUTY
POWER TILT

J4 HD

MID-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 5 mph

WEIGHT CAPACITY: 400 lbs.

TURNING RADIUS: 21"

COLORS



FDA Class II
Medical Device



Q1450

BARIATRIC FRONT-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 4.5 mph

WEIGHT CAPACITY: 600 lbs.

TURNING RADIUS: 26.5"

COLORS



FDA Class II
Medical Device



J6

MID-WHEEL DRIVE

SPECS

MAXIMUM SPEED: 4 mph

WEIGHT CAPACITY: 300 lbs.

TURNING RADIUS: 22"

COLORS



#1 FOR REHAB POWER

BUILT TOUGH

Quantum® bariatric power chairs all offer an innovative design with superior outdoor performance and superb indoor maneuverability. Our bariatric line of power chairs offers speeds as high as 5 mph and weight capacities up to 600 lbs.

All of our bariatric power chairs can be equipped with power positioning for greater comfort, such as TRU-Balance® HD Power Tilt. This seating system can assist with pressure relief. To provide additional support, Stealth® cushions and backs are available to provide lumbar support or coccyx and sacral relief. All Quantum bariatric power chairs are available with Q-Logic environmental controls featuring Bluetooth® technology.



QUANTUM®

B A R I A T R I C S

Quantum® Power Chairs are FDA Class II Medical Devices designed to aid individuals with mobility impairments.

Q6 Edge® HD
450 LBS.
WEIGHT CAPACITY



Q1450
600 LBS.
WEIGHT CAPACITY



FDA Class II
Medical Device

FDA Class II
Medical Device

	Q6 Edge® HD	Q1450	J4 HD	4Front® 2 HD
Capacity	450 lbs.	600 lbs.	400 lbs.	450 lbs.
Top Speed	4.7 mph	4.5 mph	5 mph	5 mph
Turning Radius	19.75"	26.5"	21"	24.5"
Power Elevation	Yes (10")	N/A	N/A	Yes (10")

*** Please consult the back of the catalog for detailed specifications.*



TRU-Balance® 4 Power Positioning Systems

WITH ANTERIOR TILT AND MEMORY SEATING

Maximum weight capacity of up to 300 lbs.

Up to 8 customizable memory seating positions recall the saved positions of all actuators, offering the choice between sequential or synced movement for each seat function.

Memory seating profiles can be labeled for easy recall of most used positions.

Seat elevation combined with anterior tilt provides optimal access.

Consumers may find it easier to perform ADLs such as: accessing sinks and cooktop surfaces, meal preparation and reaching items in cabinets.

TRU-Balance® 4 with Dual Actuator AFP shown on the Edge 3 Stretto®



Chest Bar



FDA Class II
Medical Device

TRU-Balance® 4 Power Recline



TRU-Balance® 4
Power Posterior Tilt



DUAL ACTUATOR POWER ARTICULATING FOOT PLATFORM

Lowers to assist with transfers

Articulation range of 0-7.25"

Elevation range of 85-165 degrees

Only available on TRU-Balance® 4

TRU-BALANCE® 4

POWER POSITIONING SYSTEMS

FEATURES

- Anterior tilt options: 10, 20 and 30 degrees
- Up to eight customizable memory seating positions
- Chest bar and knee blocks available (knee blocks and chest support required with 20 or 30 degrees of anterior tilt)
- Available on the Edge 3 Stretto®, 4Front® 2, and Edge® 3
- Optional Accu-Trac Advanced Tracking Technology
- Optional WC19 crash-test approved occupied transit kit option
- Available seat sizes: 16-22" width and 14-22" depth

POWER RECLINE

- 85° - 165° of recline
- 300 lbs. weight capacity
- Available with power tilt and power adjustable seat lift
- No increase in seat-to-floor height
- Standard shear reduction and cantilever armrests

POWER POSTERIOR TILT


- 0° - 50° of tilt
- Lower seat-to-floor height
- Unmatched adjustment and durability
- Maintains center of gravity shift

DRIVE SPEEDS IN ANTERIOR TILT

- Up to 10° of anterior tilt, up to full lift: **2 MPH**
- 10-20° of anterior tilt, up to 6" of lift: **0.5 MPH**
- 10-20° of anterior tilt, over 6" of lift: **DRIVE LOCKOUT**
- Over 20° of anterior tilt: **DRIVE LOCKOUT**

COLORS

* See website for a complete list of color options.

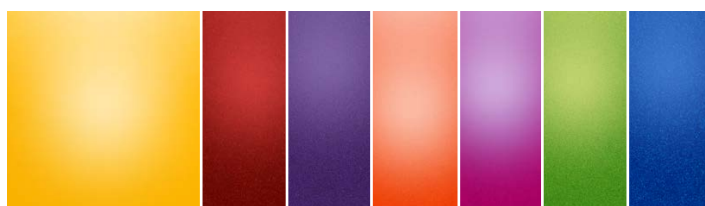

LEMON CRUSH

RED CORVETTE

DEEP PURPLE

EVOLUTION ORANGE

RASPBERRY BERET

GREEN MACHINE

BLUE JEAN

TRU-Balance® 4
with 12" iLevel®
Power Elevation





TRU-Balance®
Flex Seating



Contoured Seating



"I'M ABLE TO ELEVATE TO EVERYONE'S HEIGHT WITH iLEVEL. I FEEL MORE INCLUDED."

MORGAN STEWARD
COVINGTON, GA

Morgan Steward shown on his Stretto with TRU-Balance® 3 power positioning.

FDA Class II
Medical Device

Pediatric Seating



SEATING

AND POSITIONING



TRU-BALANCE® FLEX SEATING

No-charge static seat for the Edge 3 Stretto®, Edge® 3, Q6 Edge® 2.0, J4 and Rival®

Easily adjustable, versatile design that can grow with the user

Same ease of width and depth adjustment as TRU-Balance® 3

Seat sizes 15"-22" wide and 15"-20" deep



CONTOURED SEATING

Black vinyl captain's seat with optional armrests

Compatible with TRU-Balance 3 power articulating foot platform

Available in high back, medium back, solid seat pan, limited recline, and depth-adjustable



PEDIATRIC SEATING

Easily adjustable, this versatile design can grow with the user

Several width/depth adjustment packages start at 10"

Accepts a wide variety of after-market seats and backs

Available in numerous shroud colors that kids will love




FEATURING EXCLUSIVE
coolcore[®]
CHEMICAL-FREE COOLING TECHNOLOGY



FDA Class II
Medical Device



Convenient built-in side/front pouches

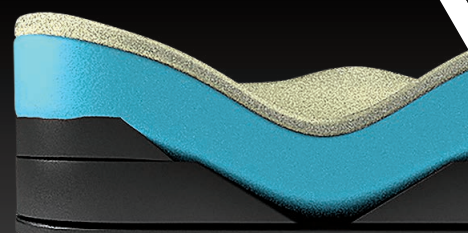

Constructed with FRG Sunmate
flame-retardant material



30+ UPF



Dry




Chemical
Free



Breathable
Comfort

"While the other makers of cooling fabrics use a number of chemicals to enhance wicking, transportation and evaporation, Coolcore fabrics are chemical free. To make that possible, we changed the essence of cooling fabric technology right down to its very core."

COOLCORE®

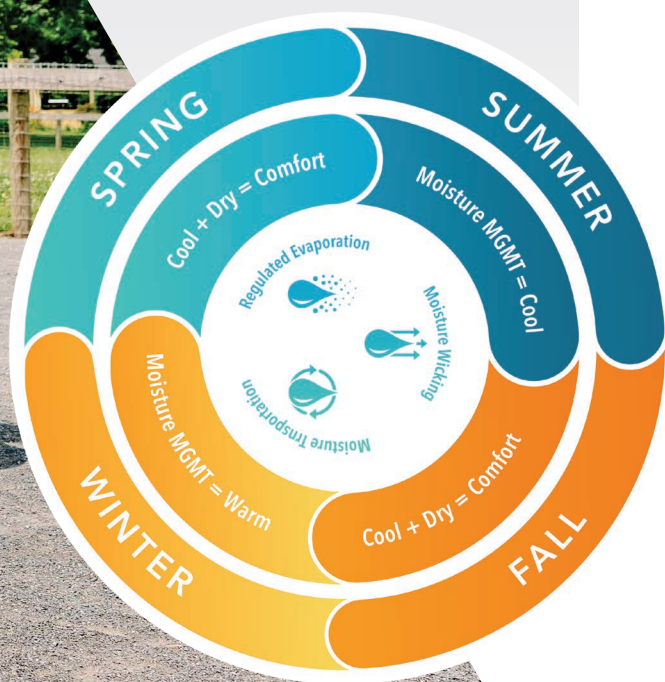


STEALTH PRODUCTS®

CUSHIONS


Stealth Products® Cushions feature patented Coolcore® technology, a chemical free, four-way stretch material that offers moisture wicking, moisture transportation, and moisture evaporation properties. Our cushions also use a color guide that identifies the suggested HCPC code and function family by a unique patch.



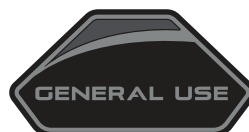

FEATURES

- Chemical free
- Regulates temperature
- Manages and distributes your body heat
- Ships out within 2 days on or off chair

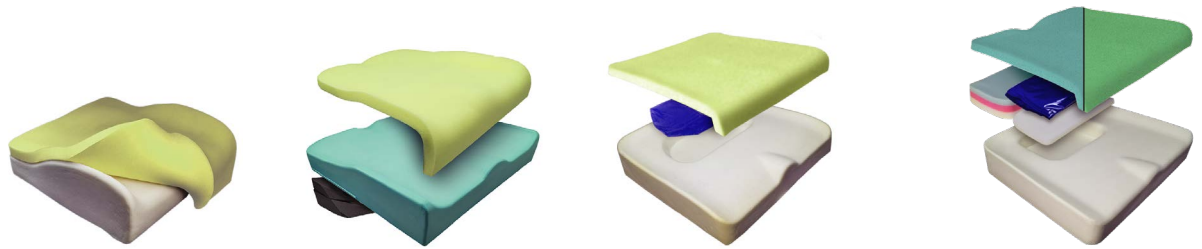

Easy-pull rubber loop
zipper handles



	TRU-Comfort 2 SPP	Essence SPP	Solution® SPP	Spectrum® Foam/Gel SPP
DESCRIPTION¹	 <p>Multi-layered foam Deep contour, CoolCore® cushion cover has “pockets” on sides</p>	 <p>Low profile Tru-Comfort 2, lateral and pre-ischial shelf Contoured, HD molded foam base, coccyx and sacral relief</p>	 <p>Standard viscoelastic foam top with twin cell aqueous viscous gel (heavier gel) insert in ischial well, mid-level contour</p>	 <p>SunMate FRG foam top (SunMate FRG XXSoft (blue); Medsoft (green)) with multi-layered foam insert or twin cell ultra-lightweight viscous gel insert in ischial well, sacral relief</p>
WEIGHT CAPACITY	14-15” W: 175 lbs. 16-21” W: 350 lbs. 22+” W: 450 lbs.	10-15” W: 175 lbs. 16-21” W: 350 lbs. 22” W: 450 lbs.	10-15” W: 175 lbs. 16-20” W: 350 lbs. 22+” W: 500 lbs.	14” W: 175 lbs. Foam & Gel 16-20” W: 350 lbs. Foam & Gel 22+” W: 650 lbs. Foam; 550 lbs. Gel
OVERALL WEIGHT	4 lbs.	3.5 - 5 lbs.	5.4 lbs.	Foam: 3 lbs. Gel: 3.5 lbs.
HEIGHT	4.5”	3.5”	3”	3”
DIMENSIONS²	14x14” up to 24x24” Odd sizes in 15-21” W SOF available	10x10” up to 22x24” Odd sizes in 15-21” W SOF available	10x10” up to 24x24” Odd sizes in 15-19” W SOF available	Foam: 14x18” up to 32x28” Gel: 14x18” up to 24x26” Odd sizes in 17” and 19” W SOF available
MSRP	\$420 - \$525	\$420 - \$625	\$394 - \$452	\$483 - \$557 (\$620 for Spectrum Foam SPP larger than 24”)
HCPSC	E2607/E2608	E2607/E2608	E2607/E2608	E2607/E2608
ALLOWANCES³	\$332.23 / \$398.98	\$332.23 / \$398.98	\$332.23 / \$398.98	\$332.23 / \$398.98

1) CoolCore® is standard on all cushions except the Simplicity G.

2) Special order sizes available for all cushions except the Glacial SP and Simplicity G. Contact sales for additional assistance.


3) Allowances listed are estimations for cushions billed in conjunction with Group 3 power chair bases using the KU modifier and may vary by state.







Stealth® ADI Q-Back 2.0

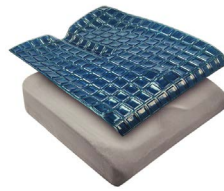FEATURES

- 450-lb. weight capacity
- CoolCore® Cover with 1.25” of Medium Soft Sunmate® FRG foam
- Two styles: AL Style (Aluminum) and AFT Style (Aluminum Flat Top)
- Available configurations: Active Contour (2.5” depth), Deep Contour (4” depth), and Extra Deep Contour (6” depth)
- Widths: 14”-22”
- Shell heights: Standard (16”), Tall (18”), Extra Tall (20”), Extra Extra Tall (22”)



Premiere P	Glacial SP	Zen SP	Simplicity G
 <p>Gray X-Soft SunMate FRG foam topper, high density positioning wedges built in for offloading.</p>	 <p>New SIM G base, fluid resistant overlay, top and sides TPU heat-sealed, low profile, lightweight cooling gel topper</p>	 <p>New SIM G base, SunMate FRG XXSoft (blue topper), low profile, lightweight</p>	 <p>High-density foam Stealth-Tek cover standard More contour to sacral/coccyx relief well</p>
14-15" W: 175 lbs. 16-21" W: 350 lbs. 22" W: 500 lbs.	10-15" W: 175 lbs. 16-21" W: 350 lbs. 22+" W: 400 lbs.	10-15" W: 175 lbs. 16-21" W: 350 lbs. 22+" W: 400 lbs.	10-15" W: 175 lbs. 16-21" W: 350 lbs. 22+" W: 400 lbs.
3.5 lbs.	3.5 lbs.	2.5 lbs.	2 lbs.
3.5"	2.5"	3"	2"
14x13" up to 22x22" Odd sizes up to 21" W SOF available	10x10" up to 24x24" Odd sizes in 15-21" W No SOF	10x10" up to 24x24" Odd sizes in 15-21" W SOF available	10x10" up to 24x24" Odd sizes in 15-21" W No SOF
\$341 - \$499	\$347 - \$462	\$331 - \$446	\$116 - \$173
E2605/E2606	E2603/E2604	E2603/E2604	E2601/E2602
\$308.53 / \$481.33	\$173.74 / \$215.97	\$173.74 / \$215.97	\$70.10 / \$136.85

DME providers are responsible for determining appropriate billing codes when submitting for insurance reimbursement. HCPCS codes provided should not be considered as legal advice and do not guarantee reimbursement. Payer coding, coverage, and bundling guidelines may apply.





FDA Class II
Medical Device



FDA Class II
Medical Device



Q-LOGIC 3 EX
ENHANCED DISPLAY



Q-LOGIC 3 EX
JOYSTICK CONTROLLER

Q-Logic 3 iAccess Programmable Switchbox
available with push buttons or toggles



Q-Logic 3 Stand
Alone Joystick



"MY EDGE 3 GIVES ME THE OPPORTUNITY TO USE ADVANCED TECHNOLOGY THAT GIVES ME SO MUCH INDEPENDENCE."

ZOE HERNANDEZ

The Q-Logic 3e features a magnetic charging port for clients with limited dexterity



Q-LOGIC 3e
JOYSTICK
CONTROLLER

Q-Logic 3 Specialty Controls
Interface Module




Q-LOGIC 3

ADVANCED DRIVE CONTROLS

The Q-Logic 3 Advanced Drive Controls System from Quantum Rehab® is like no other power wheelchair electronics system. Designed from clinician and consumer feedback, Q-Logic 3 is user-friendly and highly customizable.

FEATURES

- Wireless Bluetooth® programming through Windows devices
- Bluetooth access to computer, tablet, and phone functions
- iAccess programmable push button and toggle options for customized seating access
- Clinic Mode:
 - Quickly set up multiple input devices in client evaluations
 - Automatically assigned drive modes to input devices

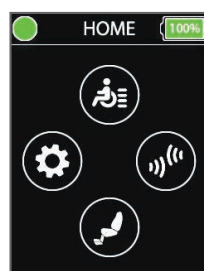

Q-Logic 3
home screen



Q-Logic 3
drive screen

ENHANCED DISPLAY

The optional Q-Logic 3 EX Enhanced Display serves as a separate screen used with specialty drive controls to display system data. With the Enhanced Display's built-in environmental controls, Q-Logic 3 can operate a wide range of appliances that use infrared (IR) remote controls, such as televisions, and serves as a Bluetooth® computer mouse emulator.



NON-EXPANDABLE

JOYSTICK OPTIONS

SIP-N-PUFF



Q-Logic NE and NE+



Q-Logic NES



Stealth® i-Drive®
Sip-N-Puff

EXPANDABLE (Q-LOGIC 3)



Q-Logic 3 EX
Joystick



Q-Logic 3e
Joystick



Q-Logic 3 EX
Enhanced Display



Q-Logic 3
Stand Alone
Joystick



Q-Logic 3 iAccess
Programmable
Switchbox available
with push buttons
or toggles



Q-Logic 3
Specialty Interface
Module

PROPORTIONAL DRIVE CONTROLS



Mo-Vis™
All-Round
Joystick



Mo-Vis™
All-Round Light
Joystick



Mo-Vis™
Multi Joystick



Stealth® Precision
Mini Proportional
Joystick



Stealth®
Mushroom
Joystick



Mo-Vis™
Micro Joystick



Mo-Vis™
Foot Control



Stealth® i-Drive®
Control Harness

MULTI-SWITCH



Stealth® i-Drive® 4 Stealth Pro,
3-Switch Head Array



Stealth® i-Drive® Tri-Array,
3-Switch Head Array



Stealth® i-Drive®
Curved Array

SINGLE SWITCH



Stealth®
Egg Switch



Mo-Vis™
Twister Basic
Button



Mo-Vis™
Twister Satellite
Button



Stealth® i-Drive®
Eclipse Tray with
Four Capacitive
Proximity Sensors



Stealth® i-Drive®
Eclipse Tray with
Four Fiber Optic
Proximity Sensors

TRAY ARRAYS

MISC. ACCESSORIES



Front and Rear
LED Fender Lights



Hydration System



Transfer Bars (Pair)



Backup Camera



Personal Item Hook




Phone Holder



Personal Items Pouch



Glove Box

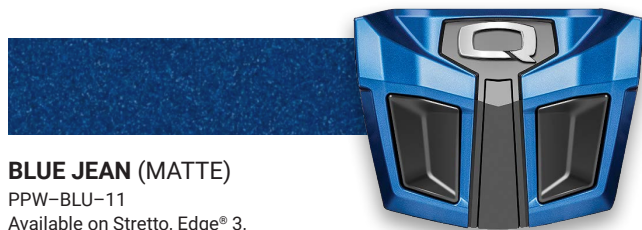

BLUE JEAN (MATTE)
PPW-BLU-11
Available on Stretto, Edge® 3,
4Front® 2, 4Front® 2 HD and
R-TRAK



RED CORVETTE
PPW-RED-01
Available on all Quantum® bases



LEMON CRUSH
PPW-YEL-04
Available on Stretto, Edge® 3/2.0/2.0X,
4Front® 2, 4Front® 2 HD and R-TRAK



GREEN MACHINE
PPW-GRN-11
Available on Edge® 3/2.0/2.0X/Z/HD,
Stretto, 4Front® 2, 4Front® 2 HD
and R-TRAK



DEEP PURPLE
PPW-PRP-05
Available on, Edge® 3/2.0/2.0X/Z/HD,
Stretto, 4Front® 2, 4Front® 2 HD
and R-TRAK



ELECTRIC BLUE
PPW-BLU-09
Available on all Quantum® bases



GREY STREET
PPW-SIL-03
Available on Edge® 3/2.0/2.0X/Z/HD,
Stretto, 4Front® 2, 4Front® 2 HD and
R-TRAK



BACK IN BLACK
PPW-BLK-01
Available on all Quantum® bases

QUANTUM® OUTBACK COLOR OPTIONS



BLACK CAMO
DGN2301319-WCM9-PCL2




WHITE CAMO
DGN2301319-WCM10-PCL2



DESERT CAMO
DGN2301319-WCM11-PCL2

COLOR OPTIONS

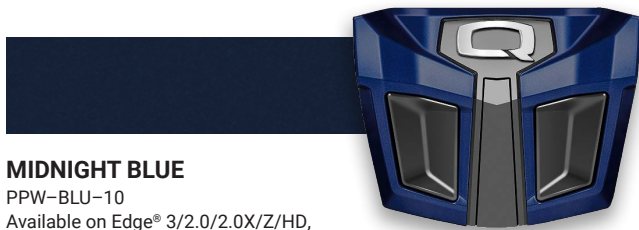

MIDNIGHT BLUE

PPW-BLU-10

Available on Edge® 3/2.0/2.0X/Z/HD, Stretto, 4Front® 2, 4Front® 2 HD and R-TRAK



EVOLUTION ORANGE

PPW-ORN-07

Available on all Quantum® bases except for J4 and J4 HD



RASPBERRY BERET

PPW-PNK-04

Available on Stretto, Edge® 3, 4Front® 2, 4Front® 2 HD and R-TRAK



PRETTY IN PINK

PPW-PNK-03

Available on all Quantum® bases except for J4 and J4 HD



NIGHT TRAIN (MATTE)

PPW-BLK-03

Available on Edge® 3/2.0/2.0X/Z/HD, Stretto, 4Front® 2, 4Front® 2 HD and R-TRAK



MILITARY GREEN (MATTE)

PPW-GRN-08

Available on Stretto, Edge® 3/2.0/2.0X, 4Front® 2, 4Front® 2 HD and R-TRAK



MILITARY DIGITAL CAMO

HYDROGRAPHICS

Available on Stretto, Edge® 3, 4Front® 2 and 4Front® 2 HD



FADED GLORY

HYDROGRAPHICS

Available on Stretto, Edge® 3, 4Front® 2 and 4Front® 2 HD




CADET

DGN2301319-TGY3-PCL2

TAN

DGN2301319-TTN1-PCL2

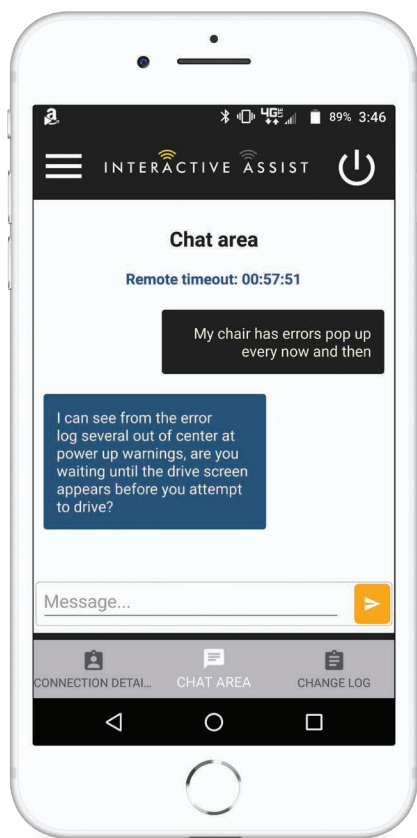

INTERACTIVE ASSIST

Reduce service visits with this innovative app.

- SCREEN MIRRORING
- REAL-TIME MONITORING
- EFFICIENCY IMPROVEMENTS
- TROUBLESHOOTING
- BATTERY MANAGEMENT
- DIAGNOSTIC HISTORY
- BASIC PROGRAMMING
- REMOTE ACCESS
- CHANGE LOG HISTORY
- SAVING XPA FILES
- MOTOR USAGE HISTORY
- TEXT CHATTING
- PROGRAMMING IR MENU
- ACCESS TO BLUETOOTH FEATURES
- DEALER-TO-DEALER PROGRAMMING ACCESS

Q-Logic 3 now features Interactive Assist. Interactive Assist from Quantum Rehab® is an app which provides a direct real-time connection from a power chair's electronics to a provider's programming station. The connection provides the technician with full system and diagnostics information along with tools, such as a real-time mirror image of the power chair's electronics display, to eliminate wasteful trips.



Download Interactive Assist today!



ECON-I

Convenience at your fingertips!

The Econ-I app is now available for download on Apple® iPhone® devices, allowing you to program Quantum® Power Chairs conveniently from your iPhone or iPad. Compatible devices include the iPhone 6S/6s Plus/SE or higher, and the iPad Air/iPad Mini 2/iPad Pro/iPad 5th generation or higher. All devices must support iOS version 12 or higher.



Download ECON-I today!

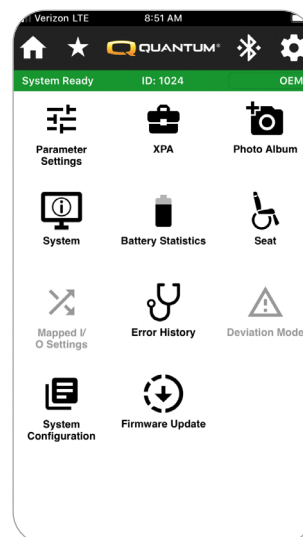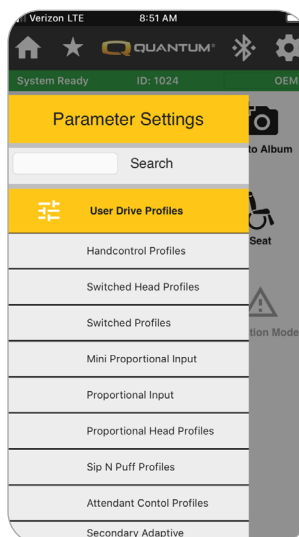


PQ-365

All Quantum® and Pride® product information combined into one app!




LIVE CHAT

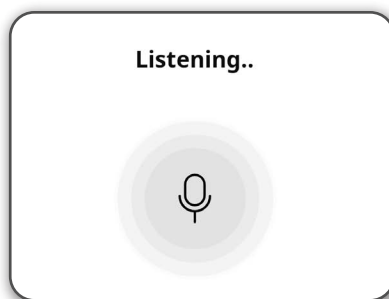


VOICE SEARCH



ZOOM FEATURE FOR
PRODUCT IMAGES

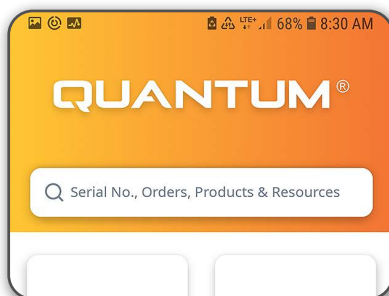

SERIAL NUMBER AND
ORDER SEARCH

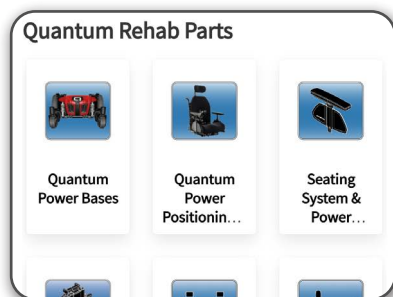

ILLUSTRATED
PARTS BREAKDOWNS

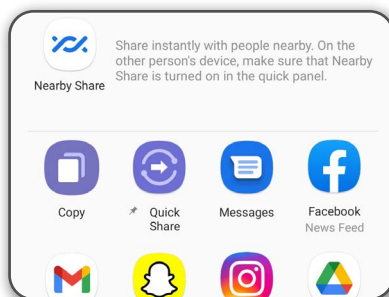


IMAGE SEND OPTION



Download PQ-365 today!

SALES SUPPORT 866.800.2002

Quantum Sales Support assists with design, ordering, pricing and quality assurance.

CUSTOM GROUP 877.465.2999

The Quantum Custom Group provides innovative custom solutions and specialized product modifications.

TECHNICAL SUPPORT EXT. 5200





Quantum Technical Support offers assistance with troubleshooting and parts order placement.

PRODUCT PLANNING & REIMBURSEMENT

Assistance with understanding Medicare codes and coverage criteria, product planning, provision guidelines, claims coding, pre-screens and assistance in regard to high-end rehab clients.

SET TO SPEC SERVICE

Set to Spec Service from Quantum Rehab® is designed to assist you in delivering Q6 Series Power Chairs to your clients. We will pre-set, to your specifications, seat-to-floor heights, seating center of gravity, armrest heights, front riggings, thoracic and/or thigh laterals.

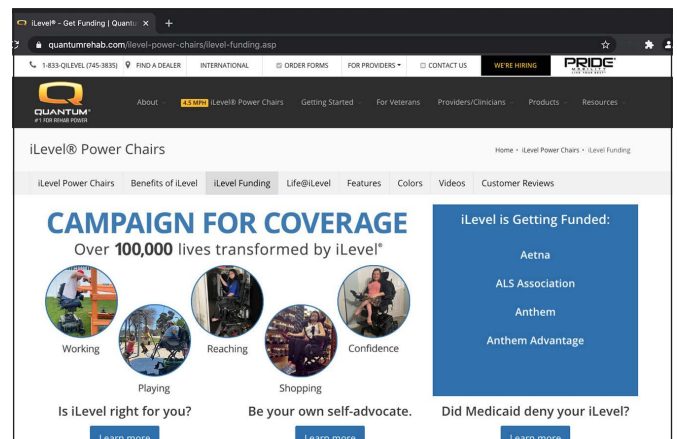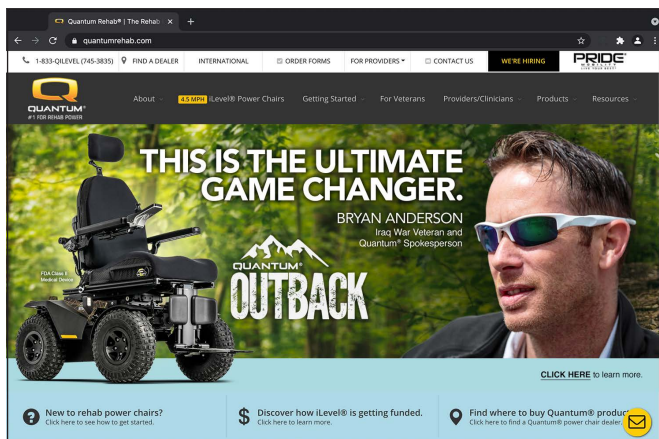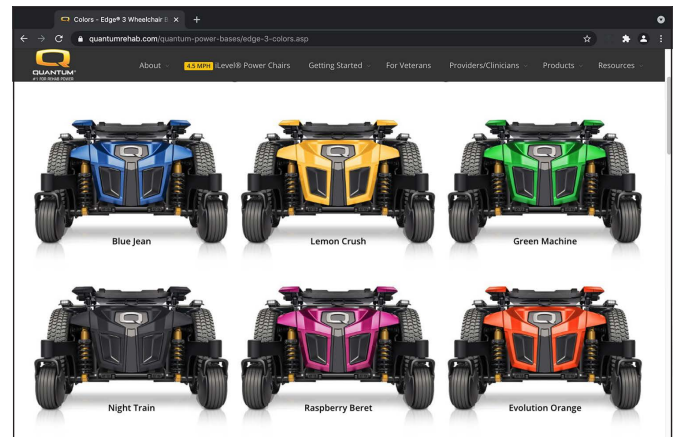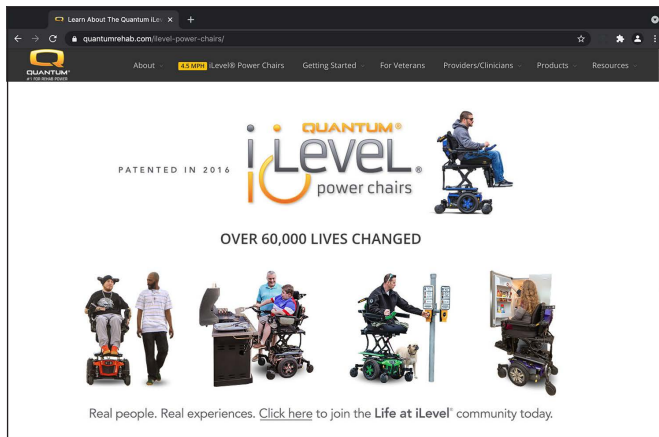

QUANTUMREHAB.COM


Quantum's official website is the ultimate source for everything you need to know about Quantum® products. Highlights include detailed product specifications, order forms, videos, user testimonials and much more.

SERVICE AND SUPPORT




LIVE LENS

Connect with technical service for remote, hands-on troubleshooting from anywhere in the field. Contact our Tech Service Dept. for details.

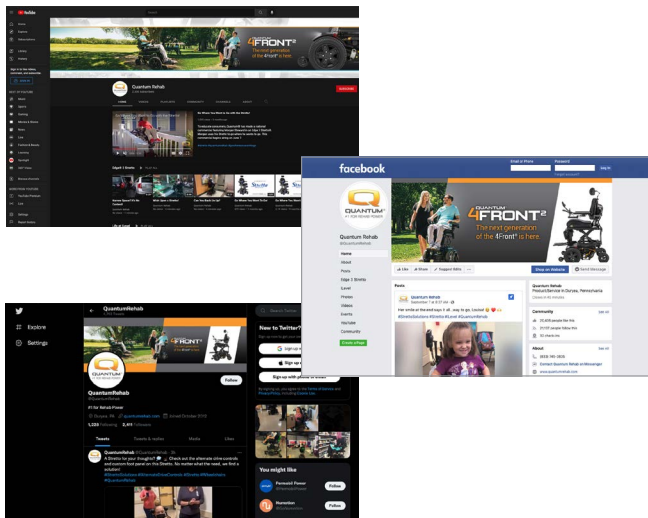

SOCIAL MEDIA

Stay up to date on the latest Quantum® events and product releases through Twitter, Facebook, YouTube, Instagram and TikTok.

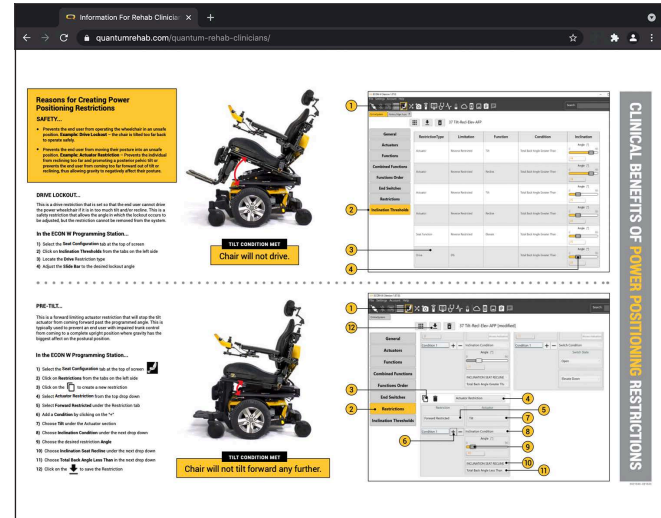

CLINICAL RESOURCES

The Clinical Resources section of the Quantum website is specifically tailored to clinicians' needs, featuring order forms, clinical references, and reimbursement information.



STEALTHPRODUCTS.COM

Stealth Products® launched in 1999, in response to a need for versatile, high-quality head and neck positioning systems, as well as switch site location hardware. Their products include: head and body support/positioning, joystick mounts, trays, seating, cushions, and pediatric mobility products.

- 1) Standard
- 2) Optional
- 3) Requires HD Suspension
- 4) Speed varies with user weight, terrain type, battery charge, battery condition and tire pressure.
- 5) Without front riggings
- 6) Battery capacity may change when adding power positioning systems.
- 7) Weight capacity is dictated by the selection of seating.
- 8) All measurements +/- 1/8" with 18" x 18" seat, fixed towers, 300 lbs. weight capacity and solid tires (add 3/8" with pneumatic tires). Depths may increase seat-to-floor height. Contact Customer Service for details. Listed measurements are in the lowest position; does not include elevating system unless specified.
- 9) 250 lbs. weight capacity with TRU-Balance®
- 10) A minimum seating size of 20" x 20" is required with iLevel® on the Q6 Edge® HD; maximum seating size is 24" x 24"
- 11) Without foot riggings
- 12) All Quantum units can be ordered with specialty controls
- 13) 6.25 mph speed available only with Q-Logic 3.
- 14) Battery weight may vary +/- 3 lbs. based on manufacturer.
- 15) Reduced to 400 lbs. with iLevel® power adjustable seat height
- 16) Seat-to-floor heights are measured with the unit unloaded. Due to the 4Front's suspension system, seat-to-floor heights are approximately 0.25" lower when the unit is loaded.
- 17) 7 mph can only be achieved with optional 4-pole motors and Q-Logic 3. Maximum speed without 4-pole motors is 6.25 mph with Q-Logic 3.

HCPCS codes provided should not be considered as legal advice and do not guarantee reimbursement. DME providers are responsible for determining appropriate billing codes when submitting for insurance reimbursement. The information contained herein is correct at the time of publication. We reserve the right to alter specifications without prior notice.

Quantum® Power Chairs are FDA Class II Medical Devices designed to aid individuals with mobility impairments.



	OUTBACK	STRETTO
HCPCS CODES For more information on HCPCS code coverage, please consult our Product Planning & Reimbursement Center.	K0899 - Power Mobility Device, Not coded by DME PDAC	Group 3: Standard (K0848/K0849), Single Power (K0856/K0857), Multiple Power (K0861)
AVAILABLE ON VA CONTRACT	Coming Soon to FSS Contract	FSS Contract V797D-70001
WEIGHT CAPACITY ⁷	300 lbs. (350 lbs. w/ Q-Captain)	300 lbs.
MAXIMUM SPEED ⁴	Up to 5 mph (6 mph with high speed package)	Up to 6.25 mph (3.5 mph with iLevel®) ¹³
GROUND CLEARANCE	4.92" (at motors)	2.9"
TURNING RADIUS ⁵	49.2"	21"
LENGTH ¹¹	42"	36"
WIDTH	28.82"	21.5" (14" drive wheels) 20.47" (12.5" drive wheels)
DRIVE WHEELS	15" pneumatic	14" or 12.5" pneumatic or flat free tires; Optional: 14" knobby
CASTER WHEELS	N/A	Front: 5" flat free Rear: 6" flat free
ANTI-TIP WHEELS	N/A	N/A
BASE WEIGHT	168.6 lbs.	126.4 lbs.
BATTERY WEIGHT ¹⁴	58.6 lbs.	40 AH: 31 lbs. each NF-22: 40 lbs. each
DRIVE WHEEL CONFIGURATION	4 Motor, Independent Drive	Mid-wheel Drive
MOTOR PACKAGES	4-pole	2-pole
BATTERIES ⁶	Group 27	40AH (2) or NF-22 (2)
BATTERY CHARGER	8A Off-board	8A Off-board
AVAILABLE ELECTRONICS ¹²	120A Q-Logic 3	120A Q-Logic 3; 75A Q-Logic NE; 75A Q-Logic NE+
SEATING SIZES		
TRU-BALANCE® 4	N/A	W: 16-22" D: 14-22"
SYNERGY®	N/A	W: 10-22" D: 10-22"
TRU-BALANCE® 3 STATIC	N/A	W: 12-22" D: 12-22"
TRU-BALANCE® 3 POWER TILT	W: 16-22" D: 16-20"	W: 12-22" D: 12-22"
TRU-BALANCE® 3 LIFT & TILT	W: 16-22" D: 16-20"	W: 12-22" D: 12-22"
TRU-BALANCE® 3 HD STATIC	N/A	N/A
TRU-BALANCE® 3 HD POWER TILT	N/A	N/A
TRU-BALANCE® 3 HD LIFT & TILT	N/A	N/A
TRU-BALANCE® 2 HD STATIC	N/A	N/A
TRU-BALANCE® 2 HD POWER TILT	N/A	N/A
TRU-BALANCE® FLEX SEAT	N/A	W: 15-22" D: 15-20"
Q4 POWER TILT	W: 16-22" D: 16-22"	N/A
SEAT-TO-FLOOR HEIGHTS^{13,16}		
TRU-BALANCE® 4 ⁸	N/A	18-19"
SYNERGY® / STATIC ⁸	N/A	16"-18" (Static only)
POWER TILT ⁸	N/A	17.5-19.5"
LIFT & TILT ⁸	N/A	18-20"
TRU-BALANCE® FLEX SEAT ⁸	N/A	16.5-18.5"
Q4 POWER TILT	N/A	N/A
Q4 LIFT & TILT	N/A	N/A
Q CAPTAIN SEAT	Captain: 19.5-21.5"	SSP: 16.5-18.5" (Flex); 18-20" (Lift) Captain: 20.5-22.5" (Flex); 22-24" (Lift)
OPTIONAL POWER ELEVATION	10" Safe Seat Elevation at 3.2 mph	12" iLevel® at 3.5 mph



EDGE® 3		EDGE® 2.0		EDGE® HD		J4	
Group 3: Standard (K0848/K0849), Single Power (K0856/K0857), Multiple Power (K0861)		Group 3: Standard (K0848/K0849), Single Power (K0856/K0857), Multiple Power (K0861)		Group 3: Heavy Duty (K0850/K0851), Single Power HD (K0858), Multiple Power HD (K0862)		Group 2: Standard (K0822/K0823), Single Power (K0835)	
FSS Contract V797D-70001		FSS Contract V797D-70001		FSS Contract V797D-70001		FSS Contract V797D-70001	
300 lbs.		300 lbs.		450 lbs. ¹⁵		300 lbs.	
Up to 6.25 mph ¹³ Optional: Up to 7 mph ¹⁷ (4.5 mph with iLevel®)		Up to 6.25 mph ¹³ (3.5 mph with iLevel®)		Up to 4.7 mph (3.5 mph with iLevel®)		Up to 5 mph	
2.7" (battery tray)		2.7" (battery tray)		3" (base frame)		2.9" (battery tray)	
20.5"		20.5"		19.75"		21"	
35.5"		35.5"		35.6"		35.5"	
24.4"		24.4"		26.25"		24.5"	
14" Solid Optional: 14" Pneumatic; 14" Knobby		14" Solid Optional: 14" Pneumatic; 14" Knobby		14" Run Flat Optional: 14" Pneumatic; 14" Knobby		14" Flat Free	
Front: 6" Solid Rear: 6" Solid		Front: 6" Solid Rear: 6" Solid		Front: 6" Solid Rear: 6" Solid		Front: 6" Solid Rear: 6" Solid	
N/A		N/A		N/A		N/A	
138.6 lbs.		138.6 lbs.		143.5 lbs.		131.6 lbs.	
40 lbs. each		40 lbs. each		52 lbs. each		40 lbs. each	
Mid-wheel Drive		Mid-wheel Drive		Mid-wheel Drive		Mid-wheel Drive	
2-pole (Optional: 4-pole)		2-pole		4-pole		2-pole	
NF-22 (2)		NF-22 (2)		Group 24 (2)		NF-22 (2)	
8A Off-board		8A Off-board		8A Off-board		8A Off-board	
120A Q-Logic 3; 75A Q-Logic NE; 75A Q-Logic NE+		120A Q-Logic 3; 75A Q-Logic NE; 75A Q-Logic NE+		120A Q-Logic 3; 75A Q-Logic NE; 75A Q-Logic NE+		75A Q-Logic NE; 75A Q-Logic NE+	
W: 16-22" D: 14-22"		N/A		N/A		N/A	
W: 10-22" D: 10-22"		W: 10-22" D: 10-22"		W: 20-28" D: 15-28"		N/A	
W: 12-22" D: 12-22"		W: 12-22" D: 12-22"		N/A		N/A	
W: 12-22" D: 12-22"		W: 12-22" D: 12-22"		N/A		N/A	
W: 12-22" D: 12-22"		W: 12-22" D: 12-22"		N/A		N/A	
N/A		N/A		W: 20-32" D: 20-25"		N/A	
N/A		N/A		W: 20-32" D: 20-25"		N/A	
N/A		N/A		W: 20-24" D: 20-25" ¹⁰		N/A	
N/A		N/A		N/A		N/A	
N/A		N/A		N/A		N/A	
W: 15-22" D: 15-20"		W: 15-22" D: 15-20"		N/A		W: 15-22" D: 15-20"	
N/A		N/A		N/A		W: 16-20" D: 16-22"	
17.5-19.25"		N/A		N/A		N/A	
Synergy®: 15.875 - 20.875" Static: 16.5 - 18.5"		Synergy®: 15.875 - 20.875" Static: 16.5 - 18.5"		16.5-18.5"		N/A	
17.5-19.5"		17.5-19.5"		18.5-20.5"		N/A	
17.5-19.5"		17.5-19.5"		18.5-20.5"		N/A	
16.5-18.25"		16.5-18.25"		N/A		16.5-18.25"	
N/A		N/A		N/A		17.25-19.25"	
N/A		N/A		N/A		N/A	
SSP: 16.5-18.25" (Flex); 17.5-19.5" (Lift) Captain: 20.5-22.5" (Flex); 21.5-23.25" (Lift)		SSP: 16.5-18.25" (Flex); 17.5-19.5" (Lift) Captain: 20.5-22.5" (Flex); 21.5-23.25" (Lift)		SSP: 16.5-18.5" (Static TB3) Captain: 20.5-22.5" (Static TB3)		N/A	
12" iLevel® at 4.5 mph		12" iLevel® at 3.5 mph		12" iLevel® at 3.5 mph		N/A	



QUANTUM®

#1 FOR REHAB POWER

- 1) Standard
- 2) Optional
- 3) Requires HD Suspension
- 4) Speed varies with user weight, terrain type battery charge, battery condition and tire pressure.
- 5) Without front riggings
- 6) Battery capacity may change when adding power positioning systems.
- 7) Weight capacity is dictated by the selection of seating.
- 8) All measurements +/- 1/8" with 18" x 18" seat, fixed towers, 300 lbs. weight capacity and solid tires (add 3/8" with pneumatic tires). Depths may increase seat-to-floor height. Contact Customer Service for details. Listed measurements are in the lowest position; does not include elevating system unless specified.
- 9) 250 lbs. weight capacity with TRU-Balance®
- 10) A minimum seating size of 20" x 20" is required with iLevel® on the Q6 Edge® HD; maximum seating size is 24" x 24"
- 11) Without foot riggings
- 12) All Quantum units can be ordered with specialty controls
- 13) 6.25 mph speed available only with Q-Logic 3.
- 14) Battery weight may vary +/- 3 lbs. based on manufacturer.
- 15) Reduced to 400 lbs. with iLevel® power adjustable seat height
- 16) Seat-to-floor heights are measured with the unit unloaded. Due to the 4Front's suspension system, seat-to-floor heights are approximately 0.25" lower when the unit is loaded.
- 17) 7 mph can only be achieved with optional 4-pole motors and Q-Logic 3. Maximum speed without 4-pole motors is 6.25 mph with Q-Logic 3.

HCPSC codes provided should not be considered as legal advice and do not guarantee reimbursement. DME providers are responsible for determining appropriate billing codes when submitting for insurance reimbursement. The information contained herein is correct at the time of publication. We reserve the right to alter specifications without prior notice.

Quantum® Power Chairs are FDA Class II Medical Devices designed to aid individuals with mobility impairments.



	J4E	J4 HD	J6
HCPSC CODES For more information on HCPSC code coverage, please consult our Product Planning & Reimbursement Center.	Group 2: Single Power (K0835)	Group 2: Single Power HD (K0837)	Group 2: Standard (K0822/K0823), Single Power (K0835)
AVAILABLE ON VA CONTRACT	Coming Soon	FSS Contract V797D-70001	FSS Contract V797D-70001
WEIGHT CAPACITY ⁷	300 lbs.	400 lbs.	300 lbs.
MAXIMUM SPEED ⁴	Up to 5 mph	Up to 5 mph	Up to 4 mph
GROUND CLEARANCE	3" (battery tray)	2.9" (battery tray)	2" (caster arms)
TURNING RADIUS ⁵	20.5"	21"	22"
LENGTH ¹¹	35.5"	35.5"	34.74"
WIDTH	24.4"	24.5"	23.19"
DRIVE WHEELS	14" Solid	14" Flat Free	10" Solid Optional: 10" Pneumatic
CASTER WHEELS	Front: 6" Solid Rear: 6" Solid	Front: 6" Solid Rear: 6" Solid	Front: 5" Solid Rear: 6" Solid
ANTI-TIP WHEELS	N/A	N/A	N/A
BASE WEIGHT	138.6 lbs.	131.6 lbs.	100.2 lbs.
BATTERY WEIGHT ¹⁴	38 lbs. each	40 lbs. each	23.4 lbs. each
DRIVE WHEEL CONFIGURATION	Mid-wheel Drive	Mid-wheel Drive	Mid-wheel Drive
MOTOR PACKAGES	2-pole	4-pole	2-pole
BATTERIES ⁶	NF-22 (2)	NF-22 (2)	U-1 (2)
BATTERY CHARGER	8A Off-board	8A Off-board	5A Off-board
AVAILABLE ELECTRONICS ¹²	120A Q-Logic 3E 75A Q-Logic NE+ (coming soon)	75A Q-Logic NE+	75A Q-Logic NE 75A Q-Logic NE+
SEATING SIZES			
TRU-BALANCE® ⁴	N/A	N/A	N/A
SYNERGY®	N/A	N/A	W: 10-22" D: 10-22"
TRU-BALANCE® 3 STATIC	N/A	N/A	W: 14-20" D: 14-20"
TRU-BALANCE® 3 POWER TILT	N/A	N/A	W: 14-20" D: 14-20"
TRU-BALANCE® 3 LIFT & TILT	N/A	N/A	N/A
TRU-BALANCE® 3 HD STATIC	N/A	N/A	N/A
TRU-BALANCE® 3 HD POWER TILT	N/A	N/A	N/A
TRU-BALANCE® 3 HD LIFT & TILT	N/A	N/A	N/A
TRU-BALANCE® 2 HD STATIC	N/A	N/A	N/A
TRU-BALANCE® 2 HD POWER TILT	N/A	N/A	N/A
TRU-BALANCE® FLEX SEAT	N/A	N/A	N/A
Q4 POWER TILT	W: 16-20" D: 16-22"	W: 20-24" D: 20-24"	N/A
SEAT-TO-FLOOR HEIGHTS ^{13,16}			
TRU-BALANCE® ⁴	N/A	N/A	N/A
SYNERGY® / STATIC ⁸	N/A	N/A	Synergy®: 15.875 - 17.875" Static: 16.75"
POWER TILT ⁸	N/A	N/A	16.75"
LIFT & TILT ⁸	N/A	N/A	N/A
TRU-BALANCE® FLEX SEAT ⁸	N/A	N/A	N/A
Q4 POWER TILT	17.5-19.5"	17.25-19.25"	N/A
Q4 LIFT AND TILT	17.5-19.5"	N/A	N/A
Q CAPTAIN SEAT	N/A	SSP: 16.5-18.5" (Flex) Captain: 20.5-22.5" (Flex)	N/A
OPTIONAL POWER ELEVATION	N/A	N/A	N/A



4FRONT® 2		4FRONT® 2 HD		Q1450		R-TRAK	
Group 3: Standard (K0848/K0849), Single Power (K0856/K0857), Multiple Power (K0861)		Group 3: Heavy Duty (K0850), Single Power HD (K0858), Multiple Power HD (K0862)		Group 3: Very HD (K0852/K0853), Single Power Very HD (K0860)		Group 3: Standard (K0848/K0849), Single Power (K0856/K0857), Multiple Power (K0861)	
FSS Contract V797D-70001		FSS Contract V797D-70001		FSS Contract V797D-70001		Coming Soon to FSS Contract	
300 lbs.		450 lbs.		600 lbs.		300 lbs.	
Up to 6.25 mph (3.5 mph when elevated)		Up to 5 mph (2.7 mph when elevated)		Up to 4.5 mph		Up to 7 mph (3.5 mph when elevated)	
3.14" (battery tray)		3.14" (battery tray)		1.5" (anti-tip wheels)		3.97" (battery tray)	
25.9"		25.9"		26.5"		25.7"	
39.9"		39.9"		38.25"		34.8"	
24"		24"		29.25"		24.2"	
14" Solid Optional: 14" Pneumatic; 14" Knobby		14" Solid Optional: 14" Pneumatic; 14" Knobby		14" Solid		14" Solid Optional: 14" Pneumatic	
Rear: 9" Flat Free		Rear: 9" Flat Free		Rear: 9" Solid		Front: 9" Solid	
Front: 4" Flat Free		Front: 4" Flat Free		Front: 4" Solid		Rear: 4" Flat Free	
130 lbs.		130 lbs.		195 lbs.		128 lbs.	
Group 24: 51.5 lbs. each Group 34: 44.8 lbs. each		Group 24: 51.5 lbs. each		52 lbs. each		51.5 lbs. each	
Front-wheel Drive		Front-wheel Drive		Front-wheel Drive		Rear-wheel Drive	
4-pole		4-pole		4-pole		4-pole	
Group 24 (2)		Group 24 (2)		Group 24 (2)		Group 24 (2)	
8A Off-board		8A Off-board		8A Off-board		8A Off-board	
120A Q-Logic 3		120A Q-Logic 3		120A Q-Logic 3 90A VR2 6-key		120A Q-Logic 3 75A Q-Logic NE+	
W: 16-22" D: 14-22"		N/A		N/A		N/A	
N/A		N/A		W: 20-32" D: 20-28"		N/A	
W: 12-22" D: 12-22"		N/A		N/A		W: 12-22" D: 12-22"	
W: 12-22" D: 12-22"		N/A		N/A		W: 12-22" D: 12-22"	
W: 12-22" D: 12-22"		N/A		N/A		W: 12-22" D: 12-22"	
N/A		W: 20-24" D: 20-24"		N/A		N/A	
N/A		W: 20-24" D: 20-24"		N/A		N/A	
N/A		W: 20-24" D: 20-24"		N/A		N/A	
N/A		N/A		W: 20-32" D: 20-28"		N/A	
N/A		N/A		W: 20-32" D: 20-28"		N/A	
W: 15-22" D: 15-20"		N/A		N/A		W: 15-22" D: 15-20"	
N/A		N/A		N/A		N/A	
N/A		N/A		N/A		N/A	
17.75 - 18.75" (Synergy® N/A)		17.75 - 18.75" (Synergy® N/A)		16.5-18.5"		16-18" (Static only)	
17.75-19.75"		17.75-19.75"		19.5-20.5"		18.5-19.5"	
17.75-19.75"		17.75-19.75"		N/A		17.25-18.25"	
16.5-18.5"		16.5-18.5"		N/A		16.5-17.5"	
N/A		N/A		N/A		N/A	
N/A		N/A		N/A		N/A	
SSP: 17.5-22.5" (Flex); 17.75-19.75" (Lift) Captain: 21.75-22.75" (Flex); 21.75-23.75" (Lift)		N/A		SSP: 18.5-20.5" (Static TB3) Captain: 22.5-24.5" (Static TB3)		SSP: 15.5-17.5" (Flex); 17.25-18.25" (Lift) Captain: 19.5-21.5" (Flex); 21.25-22.25" (Lift)	
10" Safe Seat Elevation at 3.5 mph		10" Safe Seat Elevation at 2.5 mph		N/A		10" Safe Seat Elevation at 3.5 mph	

quantumrehab.com •     
The information contained herein is correct at the time of publication.
We reserve the right to alter specifications without prior notice.

life.quantumrehab.com
(US) 866-800-2002 • (Canada) 888-570-1113



1 F O R R E H A B P O W E R

Quantum® Power Chairs are FDA Class II Medical Devices
designed to aid individuals with mobility impairments.