



Joystick Clinical Comparison



QUANTUM[®]

#1 FOR REHAB POWER

Q-Logic 3e



Q-Logic 3 EX



DISPLAY

- Button based, NO dials or toggles
- Screen is icon based (no words)
- Color LCD Screen

DISPLAY

- Can customize text or icon based profile labels
- Speed dial and On/Off toggle

FUNCTION

- Programmable soft keys
 - *iAccess type functionality where multiple pages can be programmed*
- Can operate all power actuators
Bluetooth capability to pair client's phone, tablet or computer
- Supports Interactive Assist
- Allows programming as proportional or switch operation through joystick
- Does not support specialty controls
- Recommended for end users preferring a more simplistic interface
- Internal Memory for loading of firmware pack, SD card eliminated

FUNCTION

- Speed dial vs. button speed adjust
- Toggle on/off button
- Programmable I/II buttons or option for additional iAccess add-on
- Can operate all power actuators
- Bluetooth capability to pair to client's phone or computer
- Infrared capabilities
- Supports Interactive Assist
- Programming for alternative drive control needs
- SD card required

Q-Logic 3 EX joystick CAN be replaced with Q-Logic 3E in the field with no additional harness just with proper firmware. **Firmware can be updated in the field through HHP, Econ-I or Econ-W**




Q-LOGIC 3E HARDWARE




- Magnetic charger port
- USB-C charger port
- On/Off port
- Mode port



Q-Logic 3E

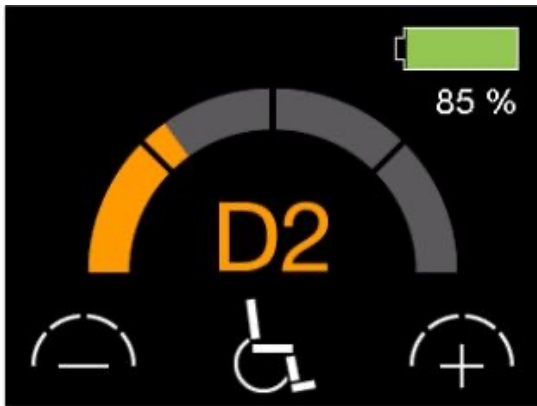

Q-Logic 3 EX

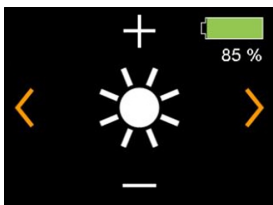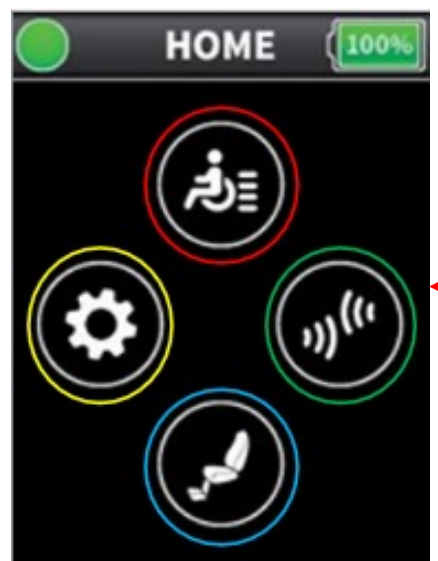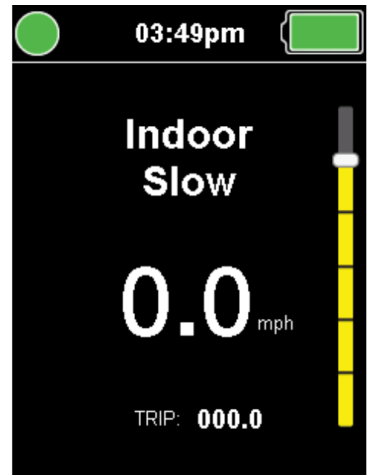


Brightness



Horn Volume



Beep Volume

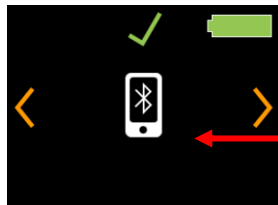

BOTH provide Bluetooth for user connectivity!

Q-Logic 3E SEATING SCREEN



Q-Logic 3EX SEATING SCREEN



SOFT KEY CUSTOMIZATION


- Essentially a built in iAccess that can have multiple pages
 - Now compatible with iAccess
- Can be programmed per profile
 - Seat Functions
 - Bluetooth hot key (takes you right to the Bluetooth screen)
 - Home screen
 - Speed up/Speed down (similar to speed pot on EX Joystick)

IACCESS MODULE

- Can have 1-19 different functions
- Single or multiple pages
- Flat or toggle buttons for:
 - Seating
 - Lighting (if LED kit)
 - Bluetooth mouse clicks etc.

Q-LOGIC 3E

- For those that need potentially all power positioning but an alternative user interface
- For those not needing functions for:
 - Alternative drive controls
- Ability to hold the “page” button to swap screen display for improved visual for indoor vs. outdoor lighting
- Compatible with iAccess

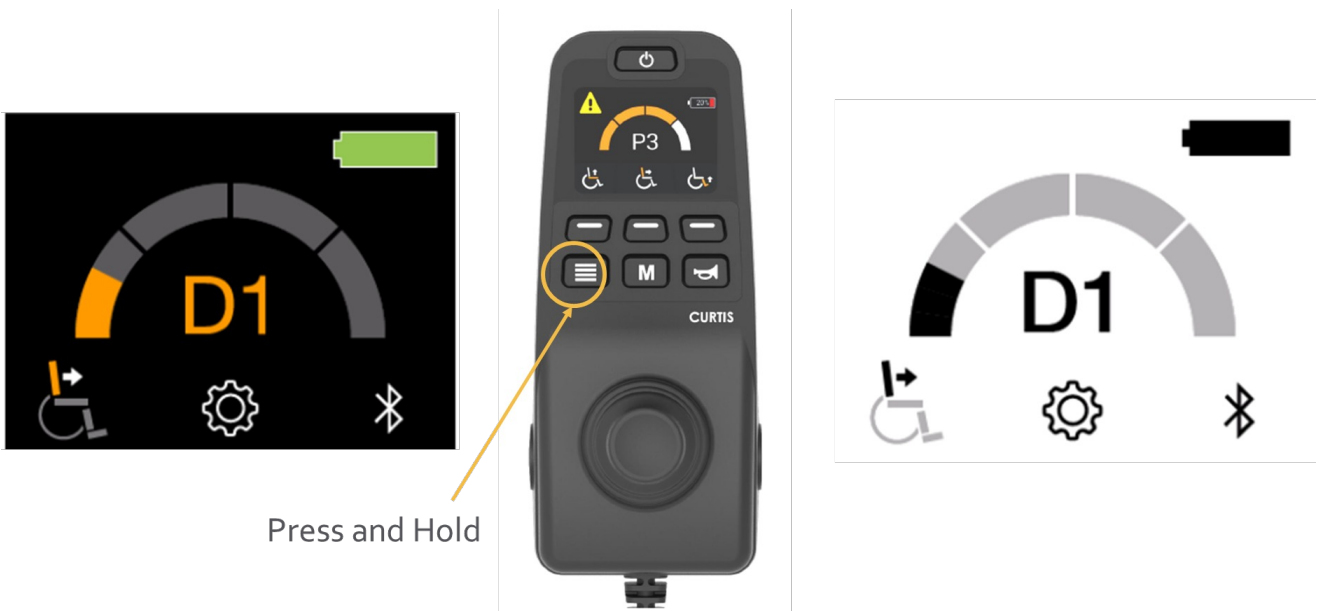

- Can do many of the same functions as Q-Logic 3 EX such as:
 - Device double command for mode changes
 - Same jack commands
 - Short, double, and long command
 - E-stop (for on/off and mode jacks)
 - Custom mapped I/O for Mode Jack ONLY
 - Now compatible with memory seating for TB4 systems

SPEED DIAL TYPES...

Profile Speed

- **Profile Speed:** the speed bar represents the total speed of the power chair.
- If multiple drive profiles are programmed in, then the speed will only highlight up a certain amount, relative to the max speed of the power chair.



Speed Pot

- **Speed Pot:** The half circle will go up all the way in each drive profile.
- Instead of being for the overall speed of the chair, the speed bar represents speed of the drive profile shown.

